


Table tray and Supporting table - HD Balance

User manual / Assembly instructions

Page 1 (2)



Assembly of the table mounts:

①

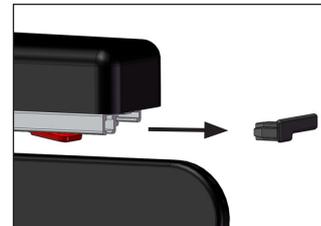




Pull out the **REAR** plastic endings from the armrest profile on each side of the wheelchair.

②

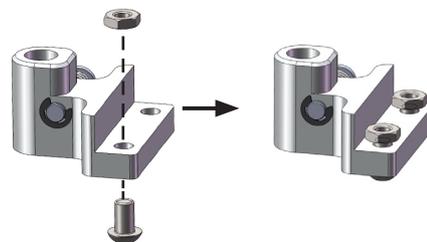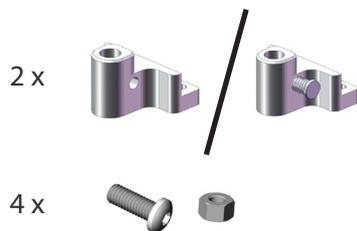



Fit the screws in the table mounts. Only secure them a few rounds.

③

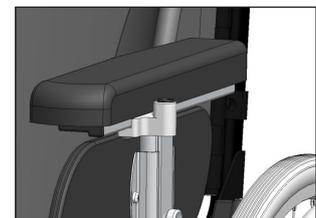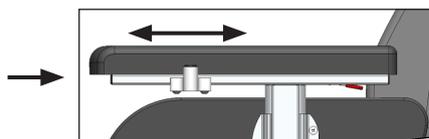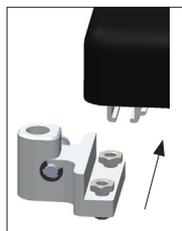


Fit the nuts, from the rear end, in the slot along the armrest profile. Fit one mount on each armrest. Move the mounts so that they sit in approximately the same location on both sides. The armrests should also be placed at the same height if the table is to sit well.

④

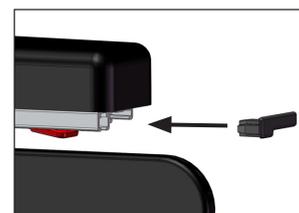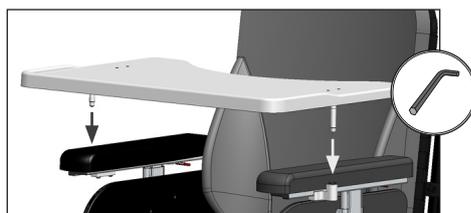



Fit the table tray in the holes on the mounts according to the picture. Position the tray, by moving the mounts along the slot in the armrest profile, to the desired position and tighten the screws. Put back the plastic endings on the armrest profile. The table tray is ready for use!
See opposite side for further adjustment instructions.

Further adjustments:

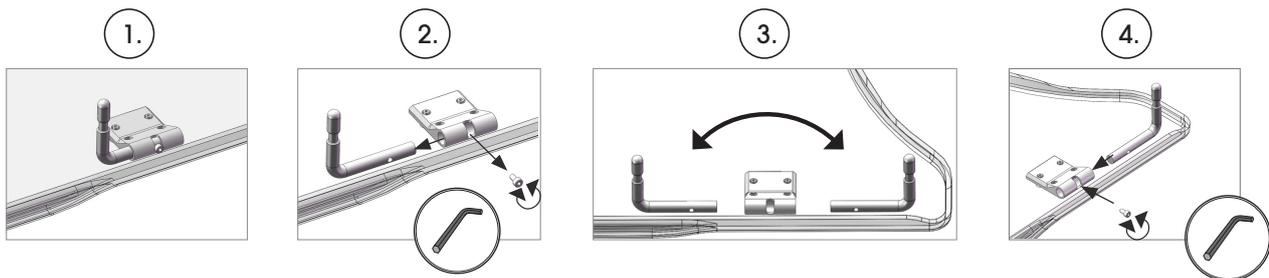

By switching side for the taps, according to the pictures above, the position of the table tray will change 85mm in depth.


Position the table tray:

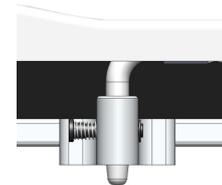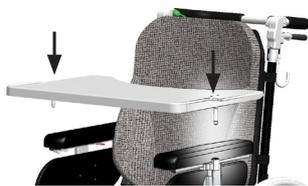

MOUNT WITHOUT LOCK: Put the table on the armrests so that the taps fit the holes in the mounts on both sides. Make sure that the taps slide down properly and that the table tray rests on the armrest.

MOUNT WITH LOCK: Put the table on the armrests so that the taps fit the holes in the mounts. Push the button (a) to make the taps slide down in correct position. Release the button to lock. Try to lift the table tray to make sure it is locked in position.

Remove the table:

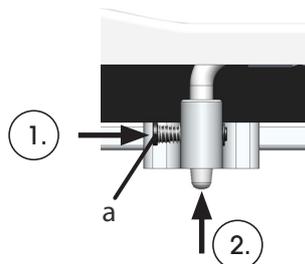

MOUNT WITHOUT LOCK: Lift the table tray upwards to release the taps from the mounts.

MOUNT WITH LOCK: 1. Push and hold the button (a). While holding the button; 2. push the taps from underneath and then lift the table tray upwards.

Use and care instructions

Tables of PC and wood are intended as a surface for the user's daily activities such as eating.

The supporting table is intended as an active support when leaning forward on the forearms, e.g. to stabilize the upper body.

The tables can be cleaned by wiping with soap and water. For further information see the Maintenance Instructions, art. no. 95730-1.



- Tables should not be used as a support for the upper body if the user suffers from hemiplegia or if there is a risk for shoulder dislocation.
- Other occurring risks are described in the wheelchair manual, art. no. 95701-1.