

# Thorasic support, Swing

## User manual / Assembly instructions

Page 1 (1)

### Parts included:

1 x




(Available in different sizes and designs)

### Assembling


Mount the support according to the pictures below.

(Requires a Thorasic support mount, see separate assembly instruction, 95808-1)

①

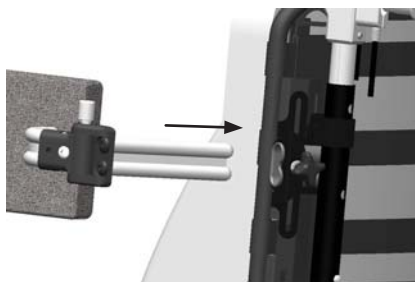

②

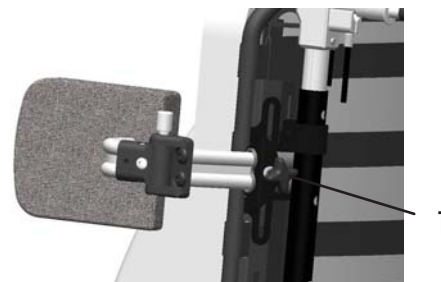

1. Slide the support into the mount according to picture 1.

2. When the support is in the desired position; tighten the knob (1) to lock it. Use a moderate force to tighten the knob, otherwise it might break.

### Adjustment of the angle


Shall be done by duly qualified person

1. First make sure that the support is in a fixed position and then loosen the screw (2) a little.

2. Secure the thorasic support in the mount, if it's not already there (see above) and swing it to the desired position.

3. Tighten the screw (2) securely to lock the support in position.

Some models can be adjusted in depth with a set screw (4) underneath the cover.

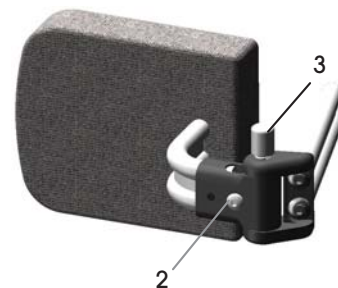

### Use

Use the button/knob (3) to unlock and swing away the support when necessary. NOTE! Try to relieve any force (from the user or cushion) from the thorasic support as the button might be hard to push otherwise.

### Care instructions

Hard parts shall be cleaned by wiping with soap and water. Covers are washed according to the instructions inside them. For further information, see the Maintenance Instructions, art. no. 95730-1.

