

# Knee joint - releasable

HD Balance

Art. no.: 62733-5 and 62734-5

**User manual /  
Assembly instructions**  
Page 1 (1)

## Parts included:

62733-5 Knee joint releasable left



62734-5 Knee joint releasable right




## Mounting

This knee joint is mounted in the same manner as the standard knee joint.

## Use

Follow the instructions below when using the releasable knee joint.

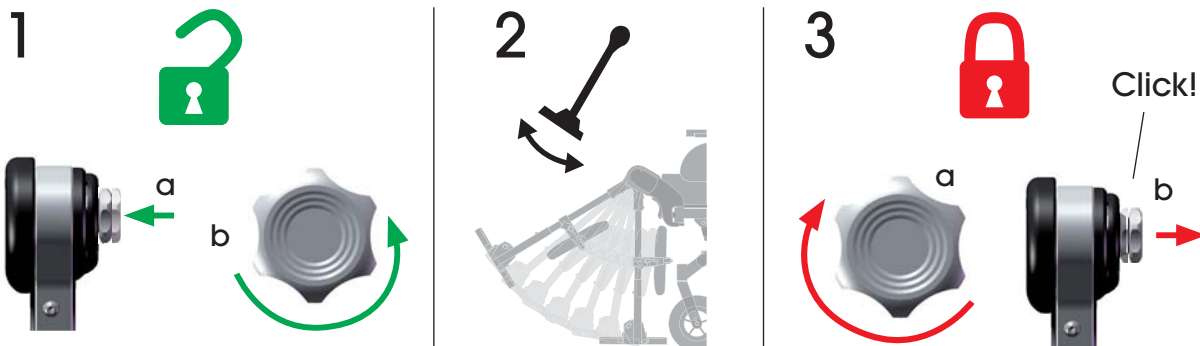

1 (a, b) Push and turn the knob counterclockwise. Now the knee joint is unlocked.

2. Adjust the leg- and footrest by unlocking the opposite knee joint in the normal way.

**⚠** Always hold the legrest securely when doing this adjustment. When the knob is unlocked the leg- and footrest fall down immediately when the button on the standard knee joint is pushed.

3 (a, b). Return to the releasable knee joint. Make sure that both knee joints are set at the same angle. Turn the knob clockwise. Pull the legrest a bit so that the knob clicks out to its outer position.

## Care instructions

Wipe with soap and water when needed. For further information, see the Maintenance Instructions 95730-1.



- The knee joint shall always be locked except when adjusting the angle of the legrests. If it's left unlocked, the structure can be damaged.