

Parts included:


Elbowrest left, art.no: 62471




Elbowrest right, art.no: 62472

**Assembling**

①

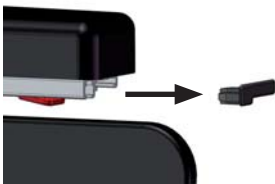

②

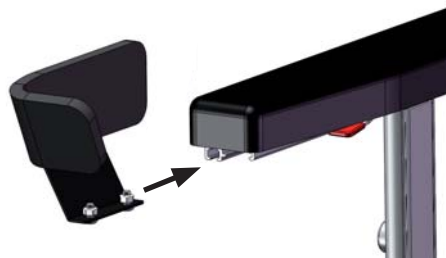

③



1. Pull out the rear plastic plug out of the armrest rail.
2. Mount screws loose and slide the elbowrest into the rail from the back.
3. The elbowrest should be placed as far back as possible. Tighten the screws in the desired position.

Use and care instructions

The elbowrest can be cleaned by wiping with soap and water. For further information see Maintenance Instructions, art. no. 95730-1.



- If the arms for some reason rest on top of the elbowrest edges for a longer period of time, harmful pressure can occur. Pay attention to this.