

Table Tray

The table trays for HD Balance are available in two materials:

- Wooden table (oak) (A), with space for the trunk.
- PC table (Polycarbonate) (B), with space for the trunk. The transparent material provides visibility for the person driving and visual body contact for the person sitting in the chair.

Both tables are available with pads for the elbows, (C and D).

All tables have a small border that prevents objects to fall off the table.



Supporting Table

The supporting table (E) is padded and can be used as an active support when leaning forward on the forearms, e.g. to stabilize the upper body.



Table brackets

Table brackets are available in two versions

- Standard (not lockable) (F)
- Lockable (G)



The tables are placed in position in the brackets which are mounted in the existing rails in the armrests.

Mug Tray

The mug tray (H) can be used as an alternative table when a smaller surface is desired. The tray has a hole for placing a cup, glass or similar. The tray is rotatable and can be swung away to facilitate movement in and out of the wheelchair. The mug tray uses a special bracket with rotation lock (I). The bracket is mounted in an existing rail on one of the armrests.




The table can be adjusted in depth in three ways:

1. Adjust the table brackets position in the arm rail (Phillips screwdriver).
2. Adjust the arm pads in depth by moving them forward / backward.
3. Turn the table pins for placing the table tray further forward. (This adjustment does not apply to the Mug Tray.)

More information about how the adjustments are made is found in the User Manual/Assembly Instructions for Table Tray and Supporting Table (art.no. 95801-1).

Dimensions:

Table 1 - Dimensions	Width [cm]	Depth [cm]	Max distance Back cushion - Table edge a [cm]
Table tray for sw38-42	61	40	37*
Table tray for sw46-50	69	40	
Supporting table for sw38-42	61	35	
Supporting table for sw46-50	69	38	
Mug tray	28	22	-

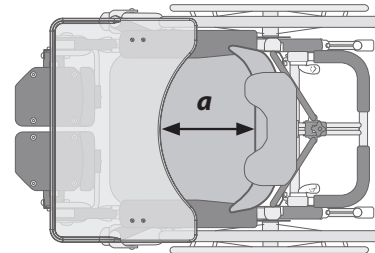

* When adjusted according to 1-3 in the numbered list above.

Keep in mind:

- Note: There are restrictions on locking users in the wheelchair against their will. It is up to the prescriber to stay informed about what regulations apply in each case.
- Tables should not be used as a support for the upper body if there is a risk for shoulder dislocation.

Article numbers

Article numbers for tables and brackets are found in the Prescription form (art.no. 95750-1).