

Side Padding

Side padding is used to further adapt the seat width to the user. Pads are attached to the insides of the armrest side supports using a pocket at the front and velcro at the back. Side padding makes the seat width 2 cm narrower (1 cm per side).

Also refer to info sheet Chassis, art.nr. 95937-1.

Side padding is not suited to Low armrests and cannot be used in combination with Wedge shaped side supports.



Profile Stops

Profile stops (B) are used to make it easy for caregivers to set a headrest or neck rest at the same height and depth each time.

Also refer to info sheet Headrests and Neck Rests, art.nr. 95932-1.



Bag Hook

Bag hooks (C) attach to the vertical portions of the push bar and are used to hang lighter bags or other items. To not interfere with the movement of the wheels, bags should be placed in the middle of the back support with a handle on each bag hook.



Keep in mind

- Tipping risk is increased if the wheelchair seat and/or back are tilted back and the bag hooks are loaded.
- Bag hooks should not be loaded with more than 5 kg total.
- Wheelchairs with increased seat or back tilting should not be equipped with bag hooks.
- Check that the wheelchair is equipped with anti-tip devices and that they are set to the active position.
- Responsibility lies with the caregiver and user to check tipping risk and ensure that anti-tip devices are used.

IV Pole

It is possible to equip HD Balance with an IV pole. This can make it easier for the user to be more mobile for certain types of treatments or examinations.

Construction is stable and consists of a strong, light-weight aluminium pole (D), and a mount (E, next page) which is attached to the back frame of the wheelchair on the left or right side. The length of the IV pole is easily adjusted with a knob. The angle is adjusted with a knob so that the IV pole can always be held upright regardless if the seat or back are tilted. The pole has two hooks so that it can carry two drip bags.



Keep in mind

- Always keep the IV pole as vertical as possible.
- Always check that the knob is tightened before use, otherwise the IV pole can fall forwards or backwards.
- Using an IV pole increases tipping risk, especially with heavy drip bags and with the seat or back tilted.
- Be aware of doorway heights when an IV pole is used.
- Always use anti-tip devices.

E

The IV pole mount is labelled with a warning symbol on the outside to highlight that special information exists. See document 95707-1.

Article numbers

All article numbers are found in the Prescription Form, art.nr. 95750-1.