

Fitting the oxygen holder

Art. No. 95101-1

Fitting the oxygen holder to HD wheelchair models HD 500 and HD 600

This document only applies to holders made by JANMEK, Tingsryd.
The holders are sold by JANMEK, Waldion AB and AGA Gas AB, among others.

We have inspected this holder and approve of it being fitted to the models of wheelchair stated above in such a way that ensures that the CE marking continues to apply.

The supplier must provide the assembly instructions stating how to fit the holder.

However, the following must also be complied with:

- The holder must be fitted as low as possible to avoid unnecessary risk of tipping.
- Extended tipping protection/bars must be used.
- Fitting this holder may affect function and prevent use of accessories.
For example, we advise against using the holder with the following accessories: bag hook, extended tilt bar and extended gas spring.
- Other necessary information must be added to the instructions for use.
- Warning signs (stickers) should be fixed to the wheelchair.

It is the fitter's/prescriber's obligation to assess whether the safety risk is excessive.
This varies from case to case. Increased risks can sometimes be compensated for, such as reducing the increased risk of tipping backwards by limiting the seat tilt and/or backrest folding function.

If the risk is judged to be excessive, the holder must not be fitted.

All the work must be carried out and assessed by competent personnel in a professional way.