

MAINTENANCE of HD wheelchairs

Art. No. 95105-1

	<p>General recommendations for maintenance. To achieve optimal safety and long use of the wheelchair, it must be kept clean and in good condition. Any defects and missing components should be dealt with and replaced immediately. The way in which the wheelchair is used affects the need for maintenance. Heavy-duty use and a lot of outdoor use naturally increase the maintenance requirements.</p>	
	How often?	Type of maintenance
1	<p>Continual</p> <p>Can be carried out by healthcare staff using the instructions for use of the wheelchair as a basis. No special knowledge is required.</p>	<p>Clean when required. Lacquered, painted and coated surfaces: – Clean using normal all-purpose detergent. Plastic components: – Clean/wipe clean and dry using normal all-purpose detergent. Fabric covers: – Wash using washing liquid for textiles. – Removable covers can be machine-washed; see the washing instructions in the covers.</p> <p>Repair any defects and replace any missing parts.</p> <p>See the instructions for use.</p>
2	<p>Periodically once or twice a year</p> <p>Can be carried out by healthcare staff using the instructions for use of the wheelchair as a basis. No special knowledge is required.</p>	<p>Examine the general condition of the wheelchair. – General cleaning. – Check all functions – even the ones not used. – Joints may sometimes need to be lubricated with a drop of oil. The tilt locking unit (see the instructions for use) may also need to be lubricated with a few drops of oil, especially if it has not been used for a long time.</p> <p>PLEASE NOTE! If lubricating the components of the disc brake system, make sure that no oil or grease ends up on the brake disc, because this will lessen the braking effect.</p> <p>Chromed bars and aluminium profiles with a telescopic function can be slightly lubricated with Vaseline or a grease approved as a foodstuff (organic and non-toxic).</p>
3	<p>Periodically every third year</p> <p>After the wheelchair is eight years old: at two-year intervals</p>	<p>Thorough inspection, reconditioning. Update of components if required. See the special reconditioning instructions. Art. No. 95107-1 NB! This must be carried out by a specialist/expert.</p>