



ASSEMBLY INSTRUCTIONS

Spoke guards for HD wheelchairs

Art.no. 15 475 24"
 15 476 20"

INSTRUCTIONS:

1. Remove the protective plastic film from both sides of the spoke guard.

2. Fit the spoke guard

- Fit the spoke guard towards the spokes on the outside of the wheel by pushing it in place behind the hand rim. Make sure that the holes (a) fit with the spokes, otherwise rotate the spoke guard into the right position. See figure 1.

3. Mount the cable ties.

- Fit the three cable ties (b), included in the package, as shown in figure 2. Fit them from the inside of the wheel so that the locking head of the cable tie does **not** end up on the outside.

NB. Do not pull the cable tie too tight before you have checked that the spoke guard is positioned correctly in the centre of the wheel.

4. Pull the cable ties

- Pull the cable tie to tighten it and cut off the remaining part, see figure 3.



Figure 1.

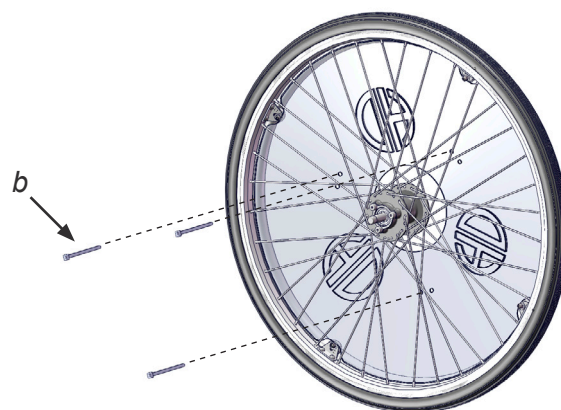


Figure 2.



Figure 3.