

ASSEMBLY INSTRUCTIONS



SEAT WIDENING ADAPTOR KIT

Art. No. **14 401 (widens 5 cm)**

Art. No. **14 402 (widens 10 cm)**

NB! The 14 402 does not fit wheelchairs with 20" or 24" wheels.

1. Secure the bar (1) inside the existing side panel fastening on the seat frame using the screw (2). Secure the fastening (3) using the four screws (4), and check that the fastening is firmly in place.
2. Insert the bar (5) into the frontal side panel fastening and secure the fastening (6) using the four screws (4). Check that the fastening is firmly in place.

Please note!

If the seat widening adaptor kit is to be fitted onto wheelchair type 500 (24" wheels), the wheel widening adaptor kit, Art. No. 14 501 must also be fitted.

The kit contains the following:

Pos.	Art.no.	Name	quantity
14 401			
1	14 460	Fastening bar, long 2	
2	81 121	Screw MRX 6x10	2
3	14 450	Fastening	2
4	81 293	Skruv K6S M6x10	16
5	14 465	Fastening bar, short	2
6	14 440	Front fastening	2
14 402			
1	14 460	Fastening bar, long 2	
2	81 121	Screw MRX 6x10	2
3	14 455	Fastening	2
4	81 293	Skruv K6S M6x10	16
5	14 465	Fastening bar, short	2
6	14 445	Front fastening	2

