

ASSEMBLY INSTRUCTIONS

TILT LOCKING UNIT, PROTECTIVE COVER

Art. No. 11 050

The protective cover for the tilt locking unit consists of a plastic cover that is designed to protect the tilt locking unit (the lock unit for the wheelchair's seat angle adjustment).

The cover can be used on all HD wheelchairs and is very easy to fit

1. Fold out the leg rests/supports as shown in Figure 1 to improve accessibility.
If the wheelchair is equipped with a complete leg frame, fold this upwards as much as possible.



fig. 1

3. Press the cover over the cross bar of the wheel frame as shown in Figure 3.



fig. 3

2. Flip up the lock catches (a) on the protective cover as shown in Figure 2.

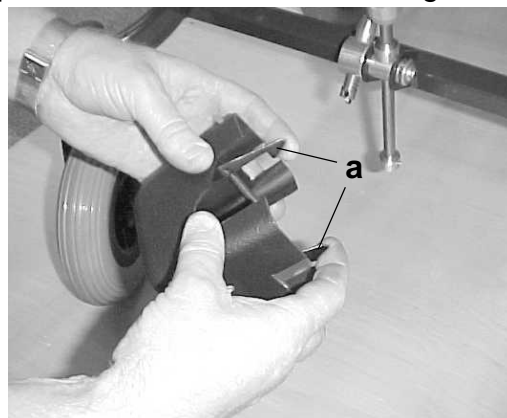


fig. 2

4. Line up the cover as shown in Figure 4 and check that the lock catches grip round the bar.

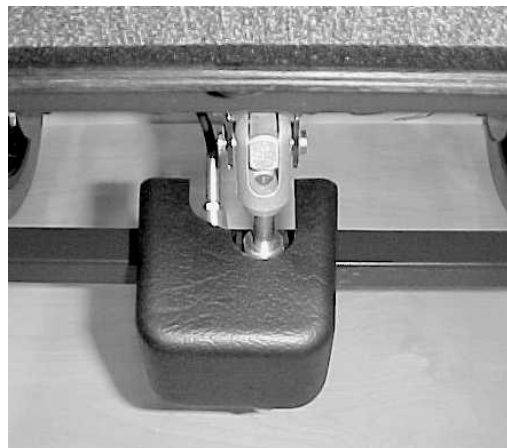


fig. 4