

ASSEMBLY INSTRUCTIONS



AMPUTEE LEGRESTS

Version: S/W 45 and S/W 38

Right and left

Amputee legrests can be fitted onto HD wheelchairs that are equipped with standard removable legrests.

The amputee legrest replaces the regular legrest and is to be attached to the wheelchair's knee joint in the same way as the standard legrest..

Fit them as follows:

1. Place the leg support bar (1) in position in the knee joint and lock it using the knob (2).
2. Slide the calf support bracket (3) onto the leg support bar and lock it into the desired position using the knob (4).
3. Fit the calf support (5) into the bracket by placing the calf support's bar (6) into the "fixing" (7) and locking it using the screw (8). The bar includes 3 holes so that it can be fitted in different positions.
4. You can fine-adjust the calf support by loosening the screw (9) slightly in the calf support brackets. Adjust the "fixing" to the desired position in the groove and tighten the screw.
5. The amputee legrest is angle-adjustable in the same way as the standard legrest, and the adjustment is made using the control knob for the leg angle on the seat frame.

Art. No. Name

<u>16 735-5</u>	AMP.Legrest, left S/W 45
<u>16 745-5</u>	AMP.Legrest, right S/W 45
<u>26 735-5</u>	AMP.Legrest, left S/W 38
<u>26 745-5</u>	AMP.Legrest, right S/W 38

The kit contains the following components:

<u>Pos</u>	<u>Name</u>	<u>Quantity</u>
1	Leg support bar	1
3	Calf support bracket	1
5	Cushion	1

