

Körbygelskydd

Bruksanvisning / Monteringsanvisning

Sida 1 (1)

Ingående delar:

2 st:



Montering

Körbygelskydden monteras på bygeln enligt bild 1. Före montering måste körbygeln tas bort från rullstolen.

1. Lossa och ta bort skruvarna (1). Se bild 1 och 2.
2. Dra loss körbygeln (2) ur fästena på ramen enligt pilarna i bild 2.
3. Blöt körbygelskydden (3) invändigt med svag tvållösning och montera dessa på körbygeln (2) enligt bild 3. Torka av överflödigt vätska.

OBS! Det kan ta mellan några timmar och ett dygn innan körbygelskydden torkar helt och sitter stadigt fast.





4. Återmontera körbygeln (2) i ramfästena med skruvarna (1) enligt bild 4.

OBS! Var försiktig när skruvarna dras åt eftersom gängorna i aluminiumet kan dras sönder.

5. Kontrollera noga så att körbygeln sitter fast ordentligt innan rullstolen tas i bruk.

Skötselråd

Körbygelskydden kan rengöras genom avtorkning med normala allrengöringsmedel eller handsprit. För övrig information se Underhållsanvisning, art.nr. 95730.

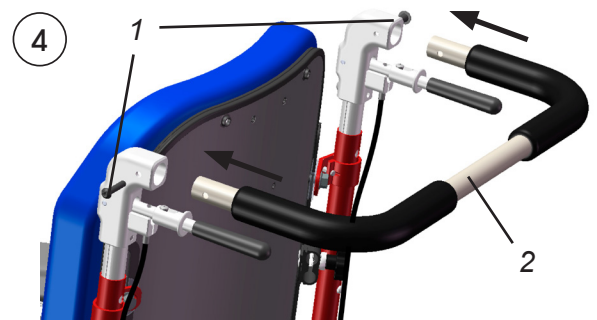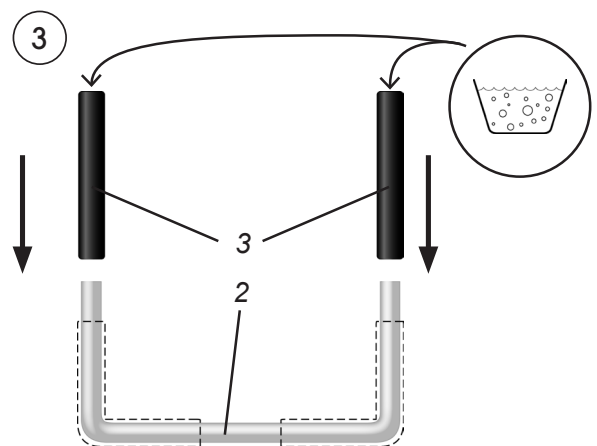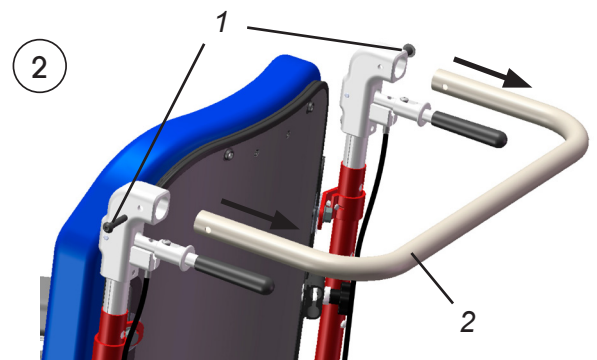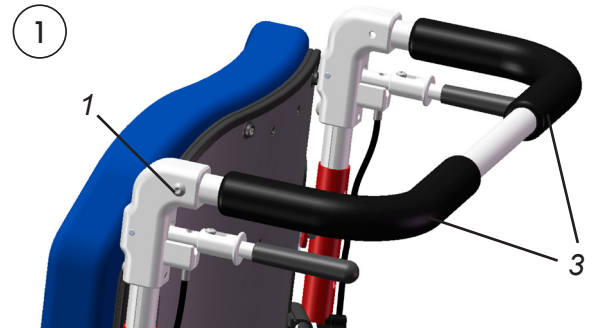

Cover for push bar

**User manual /
Assembly instructions**

Page 1 (1)

Parts included:

2 pcs:



Assembling

The covers are fitted on the pushbar as shown in figure 1. Before assembling, the push bar has to be removed from the wheelchair.

1. Undo and remove the screws (1). See figure 1 and 2.
2. Pull out the pushbar (2) from the frame according to the arrows in figure 2.
3. Use mild soap solution to wet the inside of the covers (3) and slip them on the pushbar (2) as shown in figure 3.

Note: It takes a couple of hours, or up to one day, until the covers are completely dry and firmly attached.

4. Reattach the pushbar (2) to the frame by pushing it in place and fasten the screws (1) according to figure 4.

Note: Be careful when fastening the screws since the aluminium threads are fragile.

5. Check that the push bar is properly attached before using the wheelchair.

Use and Care instructions

The covers can be cleaned by wiping with normal all-purpose cleaners or hand disinfection. For further information, see the Maintenance Instructions, art. no. 95730-1.

