

FASTENING to secure wheelchairs for transport Art. No. 13550 and 13550-5



ASSEMBLY INSTRUCTIONS

The kit contains the following components:

	Art.No 13550	Art.No 13550-5
1. Fastenings	4 pcs	2 pcs
2. Washers	4 pcs	2 pcs
3. Screws	8 pcs	4 pcs

The transport fastenings are designed to be fitted on HD wheelchairs.

Use set 13550-5 for only rear fastenings.

NB! The fastenings may only be fitted onto the wheel frame of the wheelchair.

Fit as follows:

- Place the fastening (1) onto the side bar of the wheel frame as shown in Figure 2.
- Secure the washer (2) into place using the screw (3) as shown in Figure 3 (use a wrench, size 13mm).
- Tighten securely.

By tightening the screws, the fastening is locked onto the wheel frame.

The fastening can also be fitted with the washer onto the top and inside of the wheel frame; see Figure 4.

The tensioning-strap system of the transport vehicle determines how the fastenings must be fitted.

Overleaf you will find information on suitable points at which to fit the fastenings on various models.

The tensioning straps used and the way in which these are fastened varies from vehicle to vehicle.

PLEASE NOTE!

Do not tension the straps excessively!

The tensioning straps are only designed to keep the wheelchair in place. Excessive force/pressure can result in damage to the wheelchair.

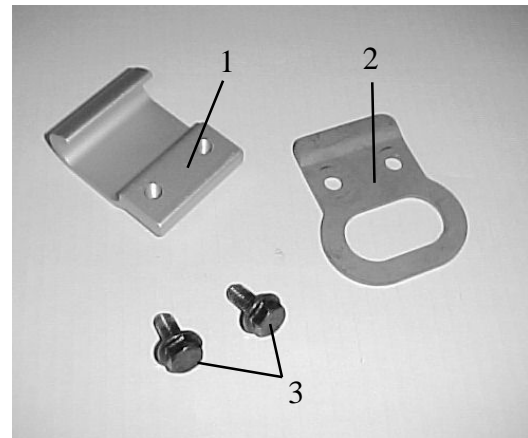


fig. 1

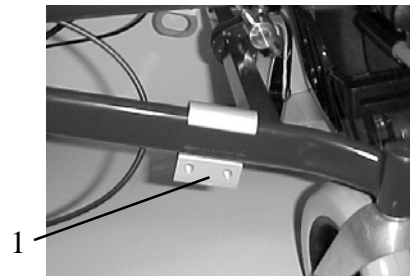


fig. 2

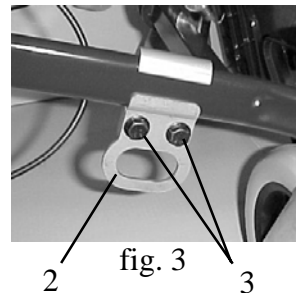


fig. 3



fig. 4

PLEASE NOTE!

These fastenings are only designed to secure the wheelchair during transport in vehicles.


Seat (safety) belts for the wheelchair user must always be attached directly in the vehicle, in the attachment points designed for this purpose.

Suitable positioning of transport fastenings

Modell HD 600

The rear fastenings (1) are placed slightly in front of the wheel attachment.
To be fitted from the inside. 1

The frontal fastenings (2) are placed slightly behind the central bar of the wheel frame.
To be fitted from the outside.

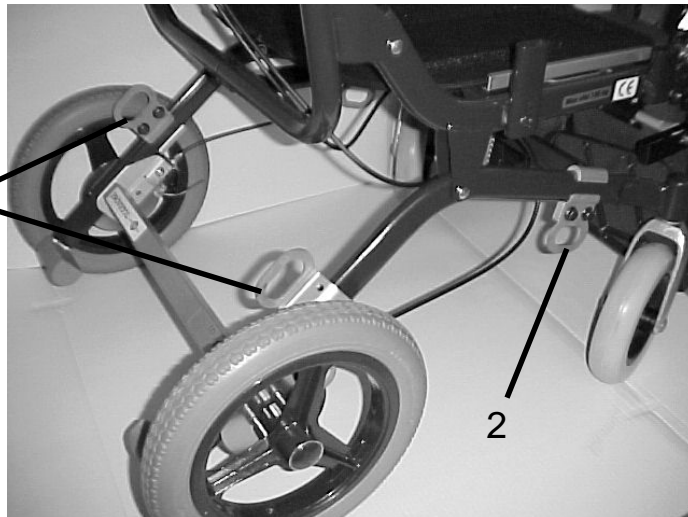

Modell HD 500

The rear fastenings (1) are placed slightly below the curved section? on the side bar of the wheel frame.
To be fitted from the inside.

The frontal fastenings (2) are placed slightly behind the parking brake.
To be fitted from the inside.



1



2

PLEASE NOTE! Do not tension the tensioning straps excessively!

The tensioning straps are only designed to keep the wheelchair in place.
Excessive force/pressure can result in damage to the wheelchair.