

Intended use

Thoracic support mount is intended to be used with an approved thoracic support from HD Rehab.

Parts included

1 x fäste



2 x bult MVBF M6x12



2 x bricka




2 x låsmutter M6



Assembly

The thoracic support mount is attached to the backrest system. Several positions are available.

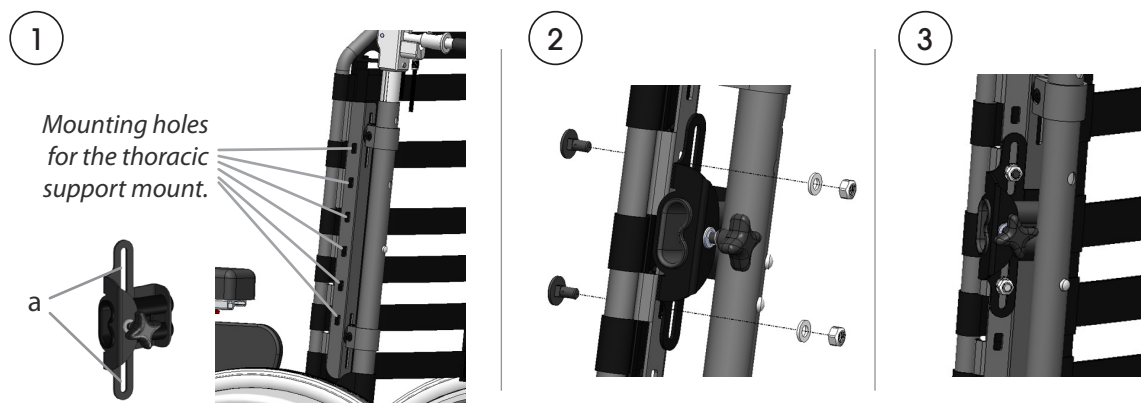

- ① Position the mount so that the slots (a) correspond to two of the mounting holes on the backrest system. Make sure that the correct height for the support is achieved by choosing suitable holes.
- ② Fix the mount according to the picture with the screws and nuts included. A 10mm wrench is required.
- ③ Small adjustment of the height can be made by loosening the screws a little and then move the mount along the slot. Tighten the screws afterwards.

Use and Care instructions

Always replace damaged parts immediately. For more information see *Maintenance Instructions*.