

Calf Strap and Calf Panel HD Balance

Art no: 63892-5, 63891-5 och 65891-5

User manual / Assembly instructions

Page 1 (1)

Parts included:

Calf Strap, art. no: 63892-5




Calf Panel, art. no: 63891-5, 65891-5



Assembling

1

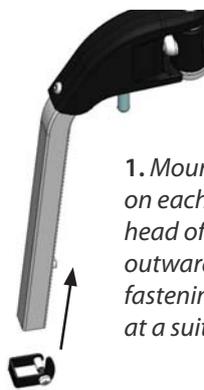

1. Mount one fastening on each tube with the head of the screw pointing outwards. Tighten the fastenings with the screws at a suitable height.

2



2. Mount the calf strap or the upper straps of the calf panel around the tube, above the fastenings. Fasten the straps on the backside with the velcro. When mounting the calf panel, also mount the lower straps around the tubes.

3a



3b



3. Finished! Adjustment of the height can be made by loosening the screws a little and then move the fastening along the tube. Tighten the screws afterwards.

Use and Care instructions

The calf strap and the calf panel are washed according to the instructions attached. For further information, see the Maintenance Instructions, art. no. 95730-1.



- The calf strap should not be used on angle adjustable legrests, as there is a risk that this can cause harmful pressure.