

Footplate Pads, HD Balance

**User manual /
Assembly instructions**
Page 1 (1)

Art. no.: 62991-1 and 64991-1

Parts included:



Assembling

Note that the size of the footplates are different depending on the seat width of the wheelchair. Sw38-42 requires the smaller size of the footplate pads (art. no. 62991) while sw46-50 requires the larger size (art. no. 64991).



1. Remove the plastic film from the velcro.
2. Place the velcro on the underside of the footplate, according to the picture. It shall match the position of the velcro at the inside of the footplate pad.
3. Mount the pad according to the picture. Push in the footplate as far as possible. Fasten the velcro.
4. Finished!

Use and Care instructions



Don't use shoes on the footplate pads.

The footplate pads shall be cleaned by wiping with soap and water. For further information, see the Maintenance Instructions, art. no. 95730-1.