

Harness Bracket - HD Balance, Flexi Back

User manual / Assembly instructions

Page 1 (2)

Art nr: 69720-1 and 69725-5

Parts included:

Harness bracket set 2 pcs : 69720-1




Strap holder, Harness bracket: 69725-5



Assembling



Assembling of the brackets:

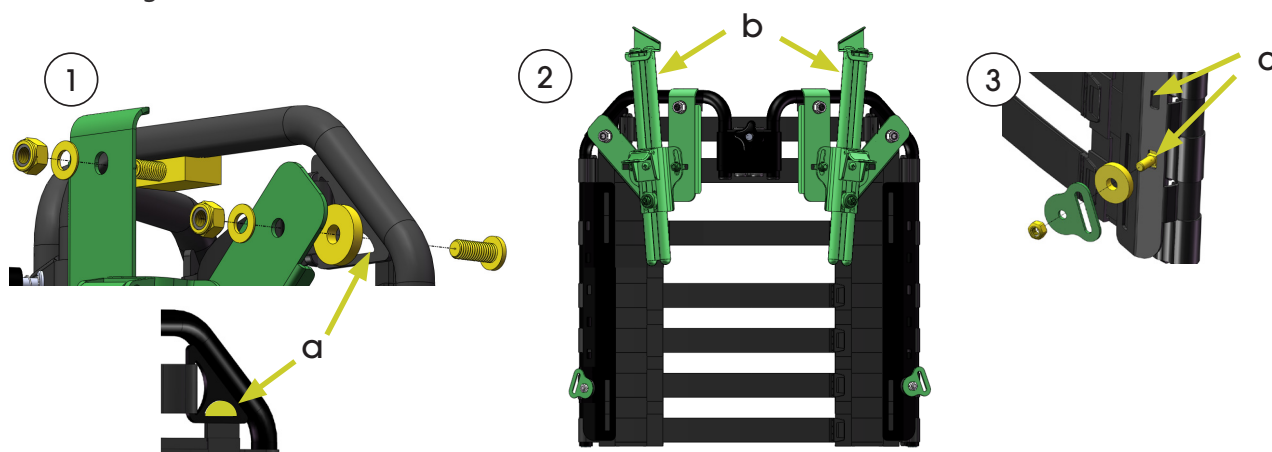

1. Install the mounting plates as shown. The outer screw is mounted in the hole marked with **a**.
Note: If the holder for the User Card and Brief Instructions is sitting in this hole, this is moved to another appropriate location on the chair. Tighten the nuts so that the plate is stable.
2. The picture shows the entire harness bracket mounted on a flexi back. Mount the extension bars (**b**) so that the strap brackets lean inwards.
3. Install the strap holder in one of the rectangular holes (**c**) on each side of the flexi back. At what height they should be attached depends on the patient and type of harness.

Use

This set of brackets is intended to be used as attachment points for different upper body harnesses. The brackets fit harnesses with a strap width of up to 25 mm. It is the manufacturer of the harness that is responsible for their product being safe to use. See the manual for each harness to see how this is positioned relative to the user. Also read "Allowed modifications with kept CE marking, HD Balance", art no 95724-1.

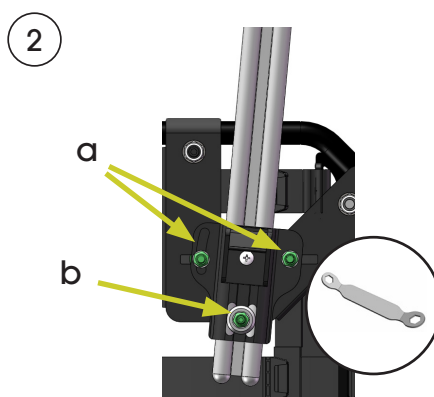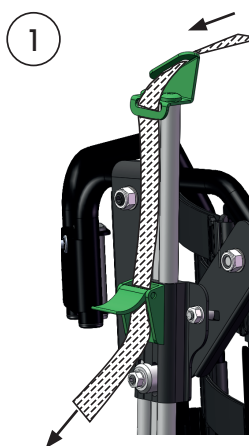

1. The upper straps of the harness are mounted according to the picture. Close the buckle to lock the strap.
2. The desired position for the upper straps is achieved by moving the extension bars vertically and laterally. This is done by using the nuts shown in the picture. Use the tool supplied with the chair. The two upper nuts (a) adjust the lateral movement and the lower nut (b) adjusts the height.



- Head- or neckrest is highly recommended if the user is considered to be at risk to hit the extension bars with their head. Pay attention so that the user does not press away the headrest, as this may cause a risk to the user.
- Be careful when adjusting the height of the extension bars as there is a risk of pinching the fingers between the bars and the flexi back.



Care instructions

The brackets shall be cleaned by wiping with soap and water. For further information, see the Maintenance Instructions 95730-1.