

Armrest coaming HD Balance

Art. no.: 62471 / 62472 / 63471 / 63472

User manual / Assembly instructions

Page 1 (1)

Parts included:


1 x armrest coaming LH

1 x armrest coaming RH




Assembly

①

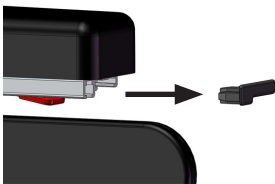

②

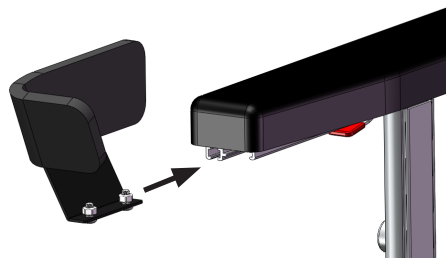

③



1. Pull out the rear plastic plug out of the armrest rail.
2. Mount screws loose and slide the coaming into the rail from the back.
3. The coaming should be placed as far back as possible. Tighten the screws in the desired position.

Use and care instructions

The coaming can be cleaned by wiping with soap and water. For further information see Maintenance Instructions, art. no. 95730-1.



- If the arms for some reason rest on top of the coaming edges for a longer period of time, harmful pressure can occur. Pay attention to this.