

# Short axle pin for castors

**User manual /  
Assembly instructions**  
Page 1 (1)

Art. no.: 67321-9 (Rebuild kit 69991-1)

## Parts included

2 x




4 x (K6S M5x10)



## Assembly

**Note:** The assembly must be performed by authorized personnel only.

Assemble the axle pins as follows:

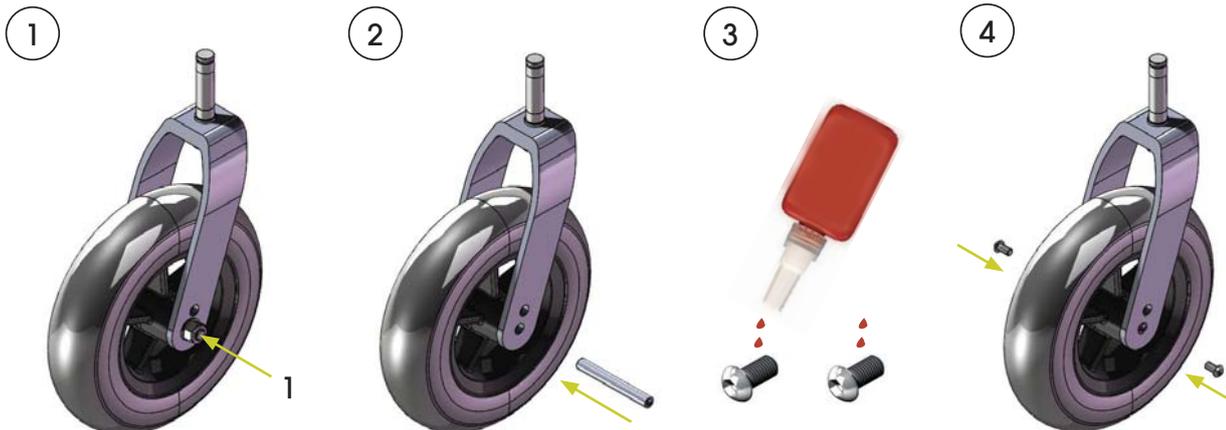

1. Disassemble the current bolt, nut, and washers (1) (The fork does not need to be removed from the wheelchair). These parts will not be re-used.

2. Fit the axle pin through the fork and castor.

3. Apply 2 drops of high strength threadlocker to each screw.

4. Drive a screw into each side of the axle pin. Make sure the screws are fully seated.

*Repeat these steps with the other castor.*

## Use and Care instructions

See the Maintenance Instructions, art. no. 95730-1. Always replace damaged parts immediately!