

Adapter thorasic support bracket - HD Balance with narrow back system


User manual / Assembly instructions

Page 1 (1)

Parts included:



Assembling

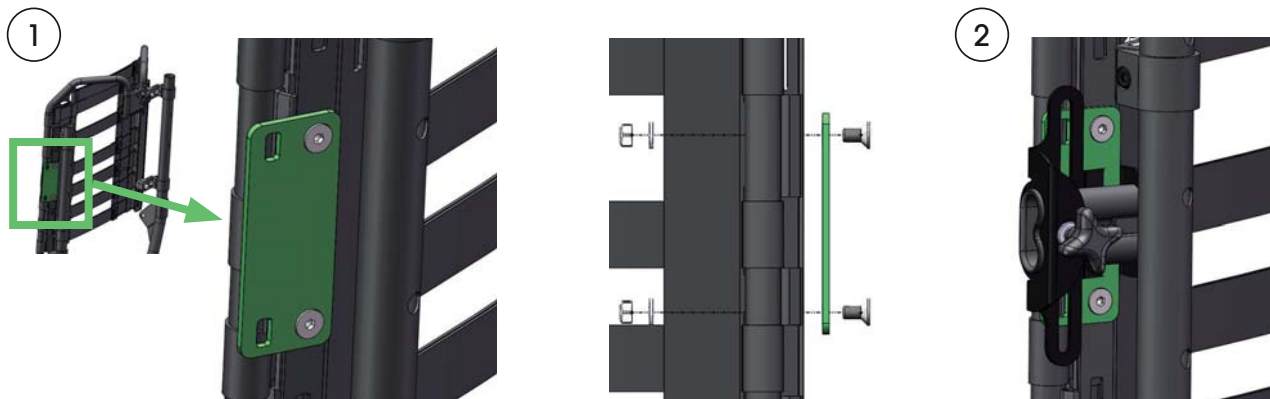

1. Position the adapter plate in the desired height along the side of the back frame. Choose suitable holes and mount with the supplied screws.
2. Mount the thorasic support bracket in the two slots on the adapter plate. Screws are supplied together with the thorasic support.

Use and Care instructions

The adapter makes it possible to use thorasic supports on HD Balance wheelchairs that have a narrow back system (-4cm). The thorasic supports will be positioned about 20mm wider compared to a back system with standard width.

Can be cleaned by wiping with soap and water. It can also be disinfected with hand sanitizer. For further information, see the Maintenance Instructions, art. no. 95730-1.