## () rehab User manual / Assembly instructions Spoke protection HD Balance, HD Force, HD Motion Art. nr. 15471, 15472, 15473 Doc. nr. 95843-1 Rev B 2023-11-03 Page 1 (1) Parts included 1 x 3 x Assembly 1 3

Remove the protective plastic film from both sides of the spoke protector.

- 1. Fit the spoke protector against the spokes on the outside of the wheel behind the hand rim. Make sure that the holes align properly with the spokes. Rotate the protector to correct the positioning if necessary.
- 2. Fit the cable ties from the inside of the wheel as shown. The locking head of the cable should be on the inside.
- 3. Pull the cable ties tight and cut off the extra material.

## Care instructions

The spoke protector can be cleaned by wiping with mild cleaning agent and water. Always replace damaged parts immediately. For more information see the Maintenance Instructions.



