

Intended use

Lockable recline angle is used to set the back recline angle in a particular position.

Parts included

1 x cable control with
lock screw, cable




1 x screw M6x22



Assembly

Note: This assembly requires technical proficiency.

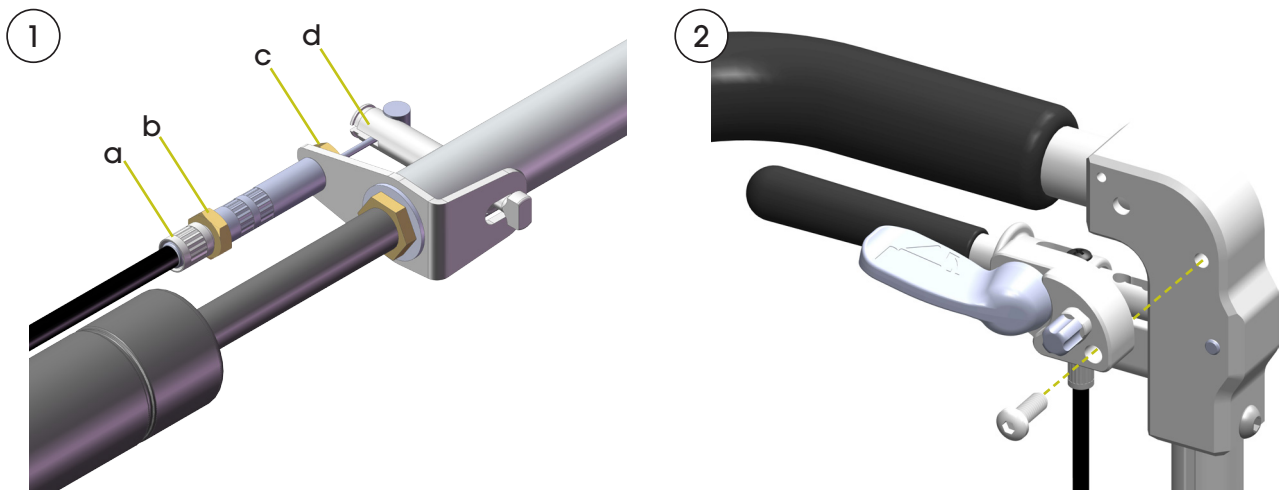


- ① Loosen the nut (b) and screw the sleeve (a) fully in so that the wire is as slack as possible. Loosen the nut (c) completely and thread the sleeve out of the plate together with the nipple connector out of the pin (d).
Remove the clips that hold the cable in place on the back frame.
- ② Remove the screw and the cable control. Use the included screw to fasten the new cable control.
- ③ Connect the cable per ① (reverse order).
- ④ Adjust the cable as follows:
Loosen the nut (b, picture ①) and screw the sleeve (a) fully in so that the wire is as slack as possible. Note that there now is some play in the controls such that the first part of the movement does not have any effect.
Screw in sleeve (a) so that the gap is almost gone and then secure with nut (b).
- ⑤ Check that the recline is functioning properly and that the cable is not tense in the locked position. The pin (d) must travel all the way back in its track when the control lever is released.

Use

Set the desired recline angle using the control. Then lock the position by screwing in the lock screw.

If a change of recline angle is desired, unscrew the lock screw. Set the new position using the control, then screw in the lock screw.

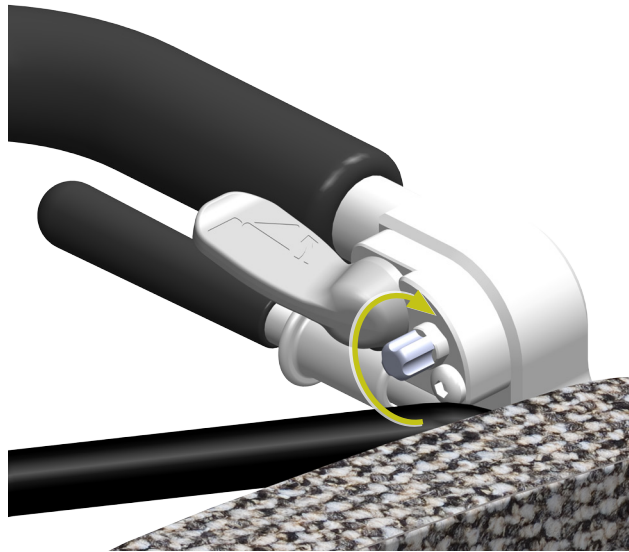

Care instructions

Always replace damaged parts immediately. For more information see *Maintenance Instructions*, document number 95730-1.