


Intended use

Padding is intended to be used with the accessory Footrest side support from HD Rehab.

Parts included

2 x Padding
(LH and RH)

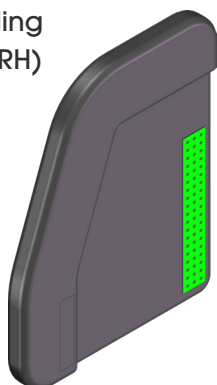


2 x velcro, soft
side 100 mm



2 x velcro, soft
side 60 mm



Assembly

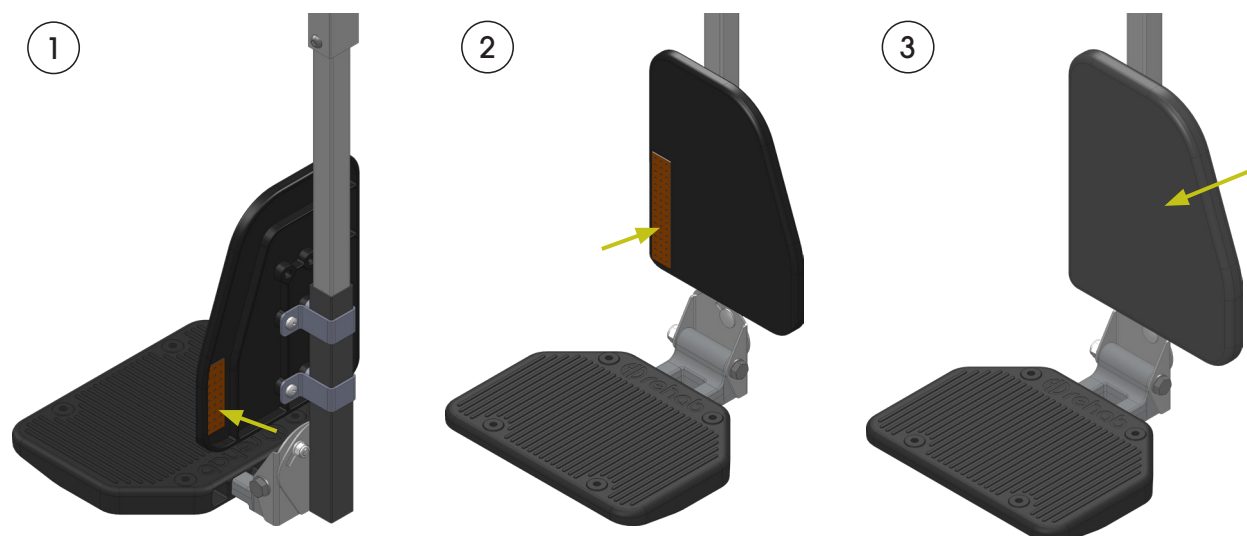

- ① Attach the shorter velcro piece to the back side of the footrest side.
- ② Attach the longer velcro piece to the front side of the footrest side.
- ③ Fit the padding over the footrest side and secure it with the velcro on both sides.
- ④ Repeat these steps for the other footrest side.

Care instructions

The footrest side padding can be cleaned by wiping with soap and water. See *Maintenance Instructions*, document number 95730-1. Always replace damaged parts immediately.