


Intended use

Footbox side padding is intended to be used with the accessory Footbox from HD Rehab.

Parts included

1 x Padding

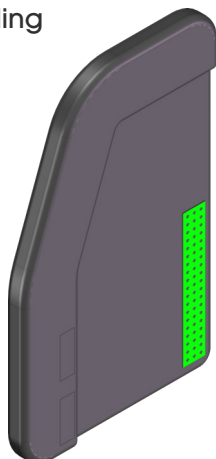


1 x velcro, soft
side 100 mm



1 x velcro, soft
side 60 mm



Assembly

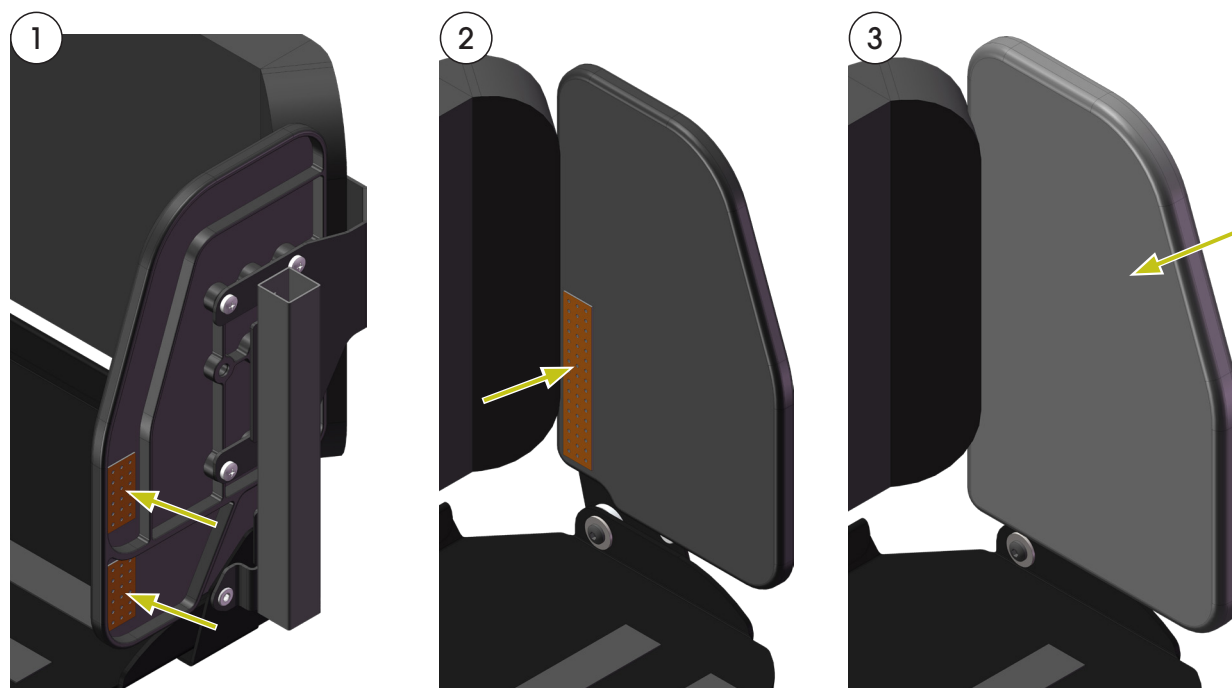

- ① Attach the shorter velcro pieces to the back side of the footbox side.
- ② Attach the longer velcro piece to the front side of the footbox side.
- ③ Fit the padding over the footbox side and secure it with the velcro on both sides.
- ④ Repeat these steps for the other side of the footbox.

Care instructions

The footbox side padding can be cleaned by wiping with soap and water. See *Maintenance Instructions*, document number 95730-1. Always replace damaged parts immediately.