

Seat cushions

Doc. nr. 95940-1 Rev D 2025-05-21

Page 1 (2)

HD Balance comes with a Standard seat cushion (A) or an Extended seat cushion (+5 cm).

The seat cushion is normally placed centred on the seat plate, so that the seat plate is fully covered. The cushion attaches to the seat via velcro strips.



Standard seat cushion

HD Balance is equipped with a contoured seat cushion of moulded foam. The cushion has a seat impression and the front portion is gently contoured for femur support. The front edge of the cushion is angled to minimise the risk of pressure against the knees.


Extended seat cushion

A seat cushion which is 5 cm longer is available to suit HD Balance with increased seat depth. The form and function are the same as the standard seat cushion.

Seat cushion measurements

Table 1 - Seat cushion measurements [cm]

Model	Depth	Width	<i>a</i> = Height front edge	<i>b</i> = Height mid back
Standard	48	38, 42, 46, 50	9	7
Extended	53			

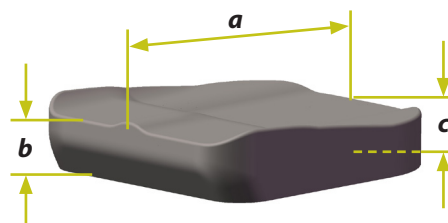

Cover fabrics

Seat cushions are covered with grey Trevira fabric (B) as standard. They can also be ordered with black Trevira fabric (C) or with black wipe-able Hygiene cover (D). All cover fabrics are removable and can be washed at 60 degrees Celsius. Covers can also be ordered separately. For more information on covers see Info sheet *Cover Fabrics*, doc. nr. 95952-1.

B



C



D



Elastic seat covers

For both standard and extended seat cushions there are elastic seat covers in black Trevira available to pull over the regular fabric. The covers provide incontinence protection and are washable at 60°C.

Keep in mind

- Seat cushions of moulded foam have a limited lifetime depending on usage. The cushions should be checked 1-2 times each year for the comfort and safety of the user. Replace threadbare and worn-out cushions as needed.

Article numbers

All article numbers are found in the Prescription Form, art.nr. 95750-1.