


# Oxygen tank holder for HD Balance


When supplemental oxygen is needed HD Balance can be equipped with an oxygen tank holder.

Oxygen tank holders are available in two models. The smaller model (A) is suited for 1 L oxygen tanks and the larger model (B) is suited to oxygen tanks of 2 to 2,5 L. The oxygen tanks used should be of the type LIV from AGA AB, or equivalent.

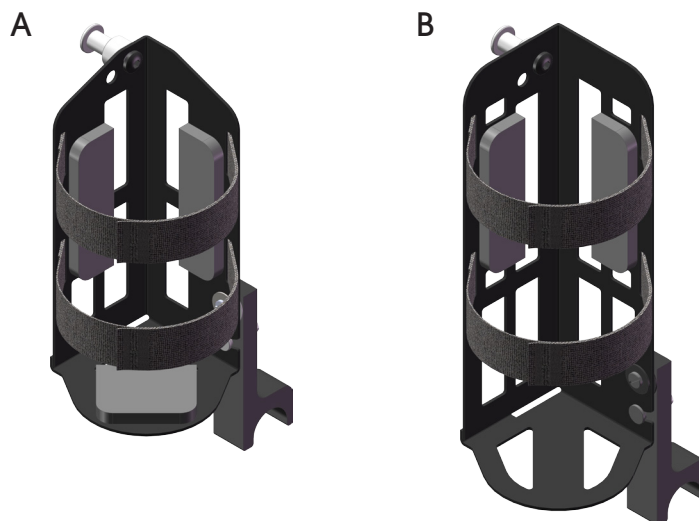

## Mount for oxygen tank holder

The oxygen tank holder is held in place via a mount (C) which is attached to the back frame of the wheelchair. The oxygen tank is then safely and easily secured in the holder with two velcro straps.

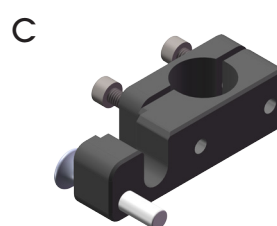

## In use

The oxygen tank holder with tank can easily be removed from the wheelchair to remain with the user when transferring out. The foot of the holder rotates up, so that the holder can stand on a flat surface.

## Measurements

Table 1 - Measurements - Oxygen tank holder

Model	Height, holder [cm]	Max diameter, tank [cm]
Small	21	12
Large	26	12

## Keep in mind

- Oxygen tanks must be handled with care and protected from shocks
- If the oxygen tank holder and tank are ever placed on the floor be aware of the risk for tipping
- Oxygen tanks should never lay flat but can be angled as far as the wheelchair allows in standard configuration (seat tilt plus back recline)
- Using an oxygen tank holder with tank increases the risk of tipping for the wheelchair, particularly when the seat is tilted and the back reclined
- Use anti-tip devices

## Article numbers

Article numbers for the oxygen tank holders and mount are found in the Prescription form, art.no. 95750-1.