

Bag Hook for HD Balance

Bag hooks (A) attach to the vertical portions of the push bar and are used to hang lighter bags or other items. To not interfere with the movement of the wheels, bags should be placed in the middle of the back support with a handle on each bag hook.

A



Keep in mind

- Tipping risk is increased if the wheelchair seat and/or back are tilted back and the bag hooks are loaded.
- Bag hooks should not be loaded with more than 5 kg total.
- Wheelchairs with increased seat or back tilting should not be equipped with bag hooks.
- Check that the wheelchair is equipped with anti-tip devices and that they are set to the active position.
- Responsibility lies with the caregiver and user to check tipping risk and ensure that anti-tip devices are used.

Article numbers

All article numbers are found in the Prescription Form, art.nr. 95750-1.