



▶ **Condensed User Manual**  
*English*

This condensed user manual is a complement to the full user manual.

Read the full manual before using the wheelchair for the first time.



## GENERAL SAFETY ASPECTS

- **Follow the maintenance instructions.** Keep the wheelchair clean and test the brakes and controls at regular intervals.
- If damage is found or components are missing stop using the wheelchair until this has been remedied.
- Use **only** approved accessories.
- The wheelchair should be operated and used with great care in order to avoid unnecessary risks.
- The wheelchair is not intended to be operated by children.
- Be aware of the risk of tipping backwards, the risk is increased if the bags or other items are hung from the backrest.
- Be aware of the user's situation in the wheelchair. There can be pinching risks with various different settings.

- Remember that the user may not always be able to communicate that a pinching situation has occurred.
- Users who are very active or who have patterns of involuntary movement can tip the wheelchair over if they grab hold of fixed objects. Watch closely.
- Clothing items that could get caught in the wheels are unsuitable for wear by the user. Take particular care with scarves and similar items, as these can tighten around the user's neck if they get caught.
- Attach the back cushion straps around the backrest system to hold the cushion in place and eliminate risk of pressure.
- When lifting: Read the instructions for lifting aids carefully and be aware of the user's arms and legs during the lift.

## TIP PROTECTORS

Tip protectors can be set in three positions with a snap lock.

To turn up the tip protectors:


1. Push the plug of the tip protector with hand or foot.
2. The tip protector is released and turns inward.
3. Turn up the tip protector to the rest position.




Always turn tip protectors back down and ensure they are locked in position.

# BRAKES

Caregiver brakes (1) and user brakes (2) are controlled as shown.

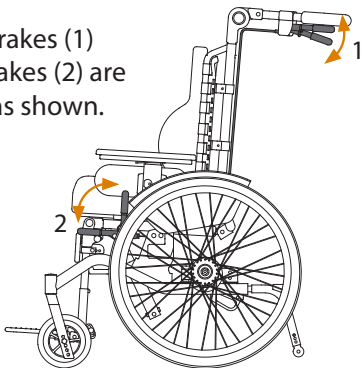

Parking brakes are activated by pushing the brake handles up (1) and then moving the locking sleeves forward (2).

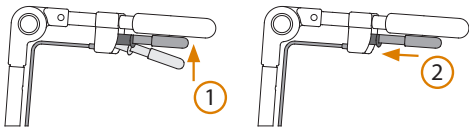

Always use both left-hand and right-hand parking brakes together.

## PUSH BAR

The push bar can be adjusted both upwards and downwards.

- Push the buttons on both the left side and the right side and move the push bar to the desired position.
- Release the buttons and check that the push bar has clicked in place.
- Check that brake and control cables run freely and are undamaged. The cables should run along the tubes of the back frame.




## ARMRESTS

Armrests are pushed in place in the mounts on each side of the seat.




### Height setting

Push in the button as indicated. Hold it in and set the desired height. The armrest post is marked in 1 cm intervals.

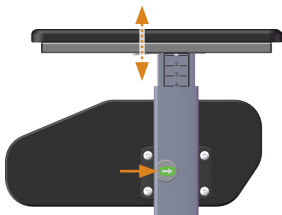

### Depth setting


Press the locking lever and set the desired position.

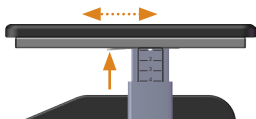

## CENTRE LEG REST


### Angle setting

Pull the knob (1) and move the leg rest to the desired position.

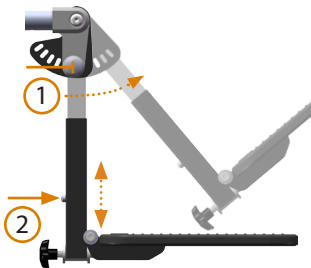

### Height setting

Push in the snap lock (2) and move to desired position.

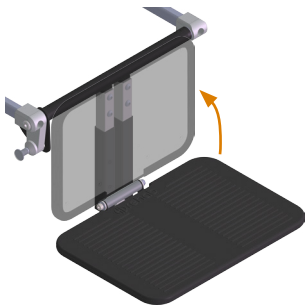

### Folding up footrest


Push the footrest plate to fold it up.



## SEPARATE LEG RESTS

### Angle setting

Pull the knob (1) and move to the desired position.

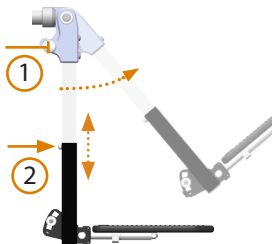

### Height setting

Push in the snap lock (2) and move to desired position.


### Folding up footrest

Push the footrest plate to fold it up to the side.



### Removal


Push the lock button (3) and pull the leg rest away.

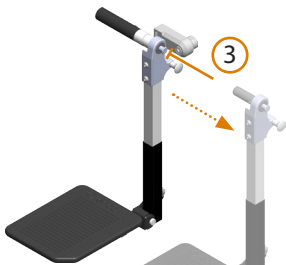

## SEAT TILT & BACK RECLINE

Seat tilt is controlled by the **green** lever on the right side of the push bar

Back recline is controlled by the **grey** lever on the left side of the push bar.

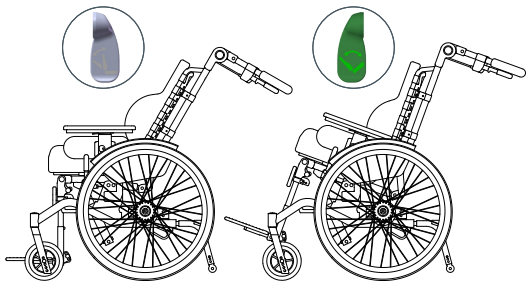

Controls are always used one at a time, never both together.

# WHEELS

## Removing wheels

Push and hold release button in wheel hub.  
Lift wheelchair slightly and pull wheel away.

## Mounting wheels

Align the wheel axle to the bushing.  
Push and hold the release button.  
Push the wheel in with the 3 pins of the hub aligned to the 3 holes in the disc brake.




Check that the wheel is secure.


# VEHICLE TRANSPORT

- Transport with wheelchair facing forward.
- Use a 4-point belt system for the wheelchair and a 3-point safety belt for the user.
- A headrest must be used.
- Remove all accessories, e.g. tray table.
- **NOTE:** Do not tension straps excessively.

Front  
anchor  
point  
around  
cross bar

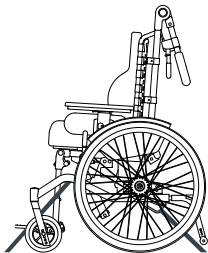

Rear  
anchor  
point  
through  
bracket

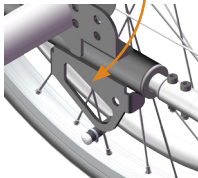


## FOLDING THE WHEELCHAIR DOWN

- Remove all accessories, the armrests, and the cushions. Tilt the seat forward.
- Hold the push bar and pull out the backrest folding pin (1) to release the gas spring. Fold the backrest forward.

## FOLDING THE WHEELCHAIR UP

- Fold the backrest up and pull out pin (1).
- Pass the pin through the hole in the gas spring and both plates of the bracket (2).

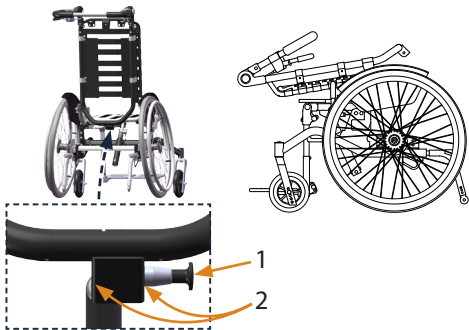

## CHECKLIST AFTER TRANSPORT

- Check that the backrest folding pin (1) goes fully through the fastening bracket (2).
- Check that the wheels are securely in place.
- Check that the tip protectors are in the correct position.
- Check the vital functions: brakes, seat tilt, back recline.



If the backrest folding pin is not correctly mounted the bracket may break and the user may be suddenly tipped backwards.



**HD Rehab AB**  
Tryffelsslingan 4  
181 57 Lidingö  
SWEDEN



+46 - 8 - 767 04 80

[info@hdrehab.se](mailto:info@hdrehab.se)



[www.hdrehab.com](http://www.hdrehab.com)



Doc. nr. 96710-1  
Condensed User Manual  
HD Motion  
Rev A 2020-09-18



Read the full user manual for more information (doc. nr. 96701-1).

## DAILY FUNCTIONAL CHECK

To ensure the safety of the user a daily check of the most vital functions should be completed, as follows:

- Test the caregiver brakes.
- Test the user brakes.
- Check that seat tilt and back recline are functioning properly.
- Check that wheels are securely attached and undamaged.
- Check that cushions are properly positioned and clean.
- Check that tip protectors are set to a suitable position.