

### Intended use

Spacer mount is used to mount a Knee Spacer or Leg Spacer on HD Motion.

### Parts included

1 x mount



4 x screw


MFX M5 x 10



### Assembly

Remove the seat cushion. Note the position of the seat plate per the scale on the right side, then move the seat plate to the outermost position.

Screw the spacer mount in place with the included screws as shown. Return the seat plate and the seat cushion to their original positions.

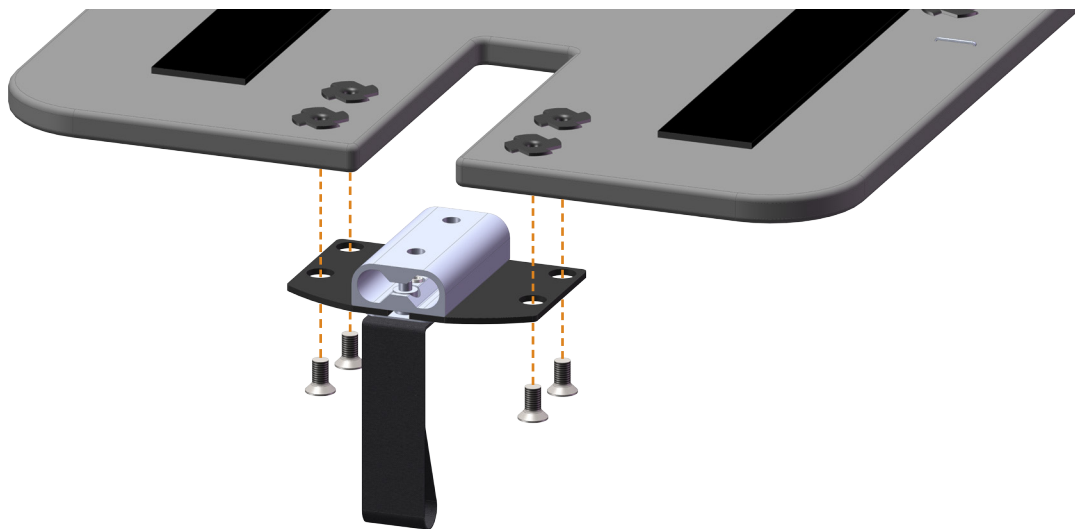

### Use and Care instructions

Always replace damaged parts immediately. For more information see *Maintenance Instructions*, document number 96730-1.