

Belt mounts

Doc. nr. 96912-1 Rev B 2021-07-06

Page 1 (1)

All positioning belts are intended to be used with belt mounts. Belt mounts for HD Motion are available in a kit with both forward and rear mounts. Each forward mount (A) consists of a rail with several attachment points where the mount for the belt strap can be placed. Each rear mount (B) has two slots to accommodate different belt strap widths.




Belt mounts are attached at specific positions on the seat frame, see 1 and 2 in picture C and the Assembly Instructions *Belt mounting kit*, doc. nr. 96802-1.

2-point belt

The belt is attached to the seat frame by the belt mounts, with either a forward placement (1 in picture C) or a rear placement (2 in picture C).

4-point belt

The belt is attached to the seat frame by two belt mounts on each side, a forward and a rear placement. See 1 and 2 in picture C.

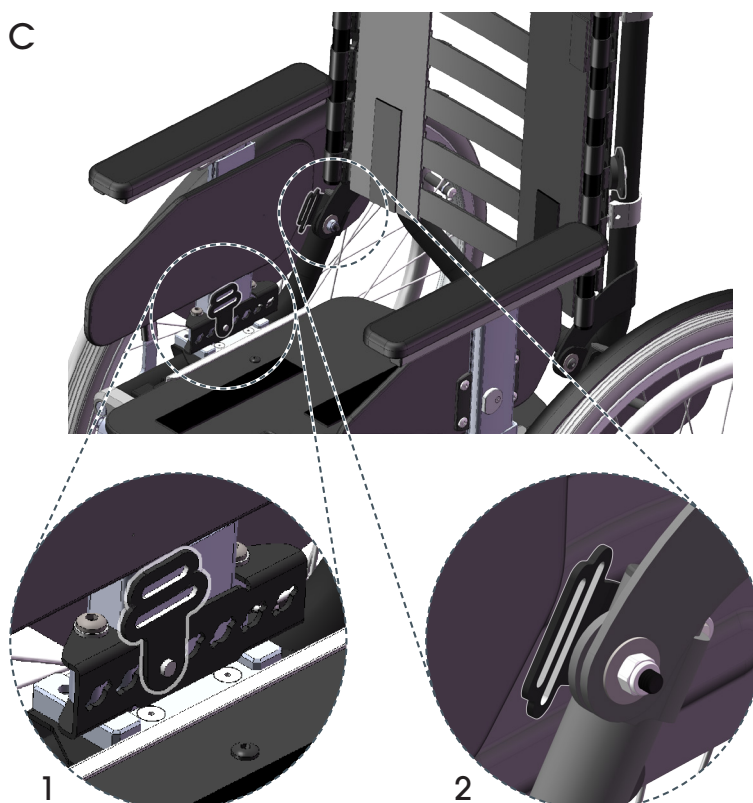

Article Numbers

All article numbers are included in the Prescription Form, document number 96750-1.

Also read *Changes permitted while maintaining CE-marking*, doc. nr. 96724-1, and *List of combination agreements*, doc. nr. 95232.