

Seat cushions for HD Motion (A) come in four sizes to suit the four different chassis sizes. The seat cushion is normally placed centred on the seat plate, so that the plate is fully covered. The cushion is held in place with velcro.



Seat cushion

The seat cushion for HD Motion is constructed with several layers of foam, each with a different firmness. It has a soft seat depression and a gentle hump in the middle at the front to separate the user's legs. The cushion also has supporting sides for the user's thighs.

Covers

Seat cushions are fully covered in stretchy black velour. The pile of the fabric on the seating surface is aligned to hinder sliding forward and to facilitate moving sideways and back.

The cover is removable and washable at 60° C. Covers can also be ordered separately.

Keep in mind

- Seat cushions have a limited lifespan depending on use. Cushions should be checked regularly for the comfort and safety of the user. Cushions should be replaced when they become worn and flattened.

Article Numbers

All article numbers are included in the Ordering Guide, document number 96750-1.