


Bag hooks attach to the back frame tubes and are used to hang lighter bags and similar items. To not interfere with the movement of the wheels, bags should be placed in the middle of the back support with a handle on each bag hook.

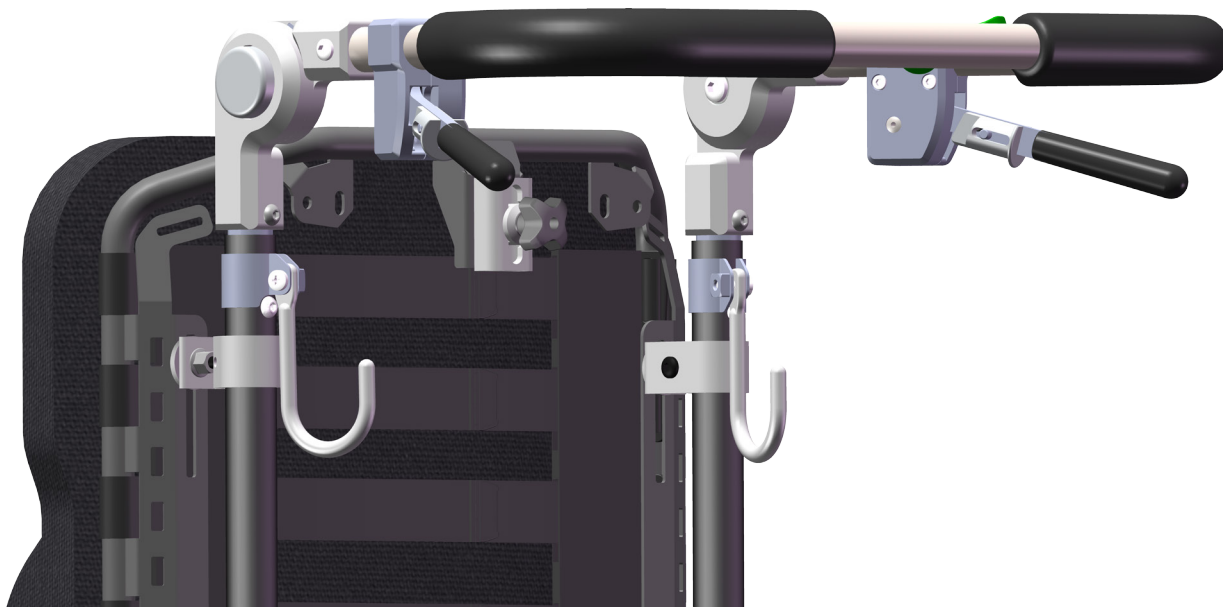

Keep in mind

- Tipping risk is increased if the wheelchair seat and/or back are tilted back and the bag hooks are loaded.
- Bag hooks should not be loaded with more than 5 kg total.
- Wheelchairs with increased seat tilt or back recline should not be equipped with bag hooks.
- Check that the wheelchair is equipped with tip protectors and that they are in the active positions.
- Responsibility lies with the caregiver and user to check tipping risk and ensure tip protectors are in place.
- Always take care that wheelchair components, particularly cables, are not damaged when using bag hooks.

Article numbers

All article numbers are included in the Ordering Guide, document number 96750-1.