

HD Motion can be equipped with a canopy (A) that folds down to provide protection from rain or sunshine.

The canopy consists of a folding fabric cover on a metal frame and is mounted on the back frame via mounts on each side. See Assembly instructions, document number 96803-1.



The mounts of the canopy are can be placed at a height suitable to the user. The mounts remain attached to the back frame while the canopy itself can be lifted off as needed.

The canopy can be complemented with a back piece (B) if more protection from behind is desired, for example to prevent sunlight coming in. The back piece is held in place with velcro. Using the canopy without the back piece allows the person pushing the wheelchair a better view of the person sitting.



Measurements

(All measurements in cm)

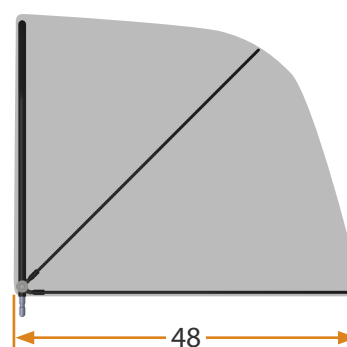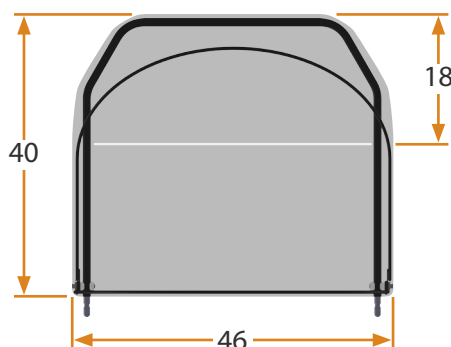

Fabric

The fabric of the canopy is wind and water resistant, with UV protection. Follow the washing instructions so that the canopy will retain these properties. When minor cleaning is required, wipe off the canopy with a damp cloth. Avoid cleaning fluids.

Article numbers

All article numbers are included in the Prescription Form, document number 96750-1.